

# Vetcorder 1.9 Update – Instructions

How to install the update for your Vetcorder

## Getting Started

The Vetcorder Software Update requires a Windows PC and the Vetcorder’s USB cable. If there are any questions, please call (844) Vetcorder (838-2673) or email [support@sentierconnect.com](mailto:support@sentierconnect.com).

**NOTE: SAVE YOUR VETCORDER FILES BEFORE UPDATING**

## 1: Save the Update Software

Go to <http://www.sentierconnect.com/vetcorder19> and click on the button that reads “Vetcorder 1.9 Updater”. Choose “Direct Download” and note where the file is located once downloaded. You will open it after connecting the Vetcorder.

## 2: Connect the Vetcorder


Using the same USB cable that is used for charging, connect the Vetcorder to your computer. Then, on the Vetcorder, follow these steps:

- a. Go to Settings
- b. Select “General”
- c. Select “Software Update”

## 3: Open the Update Software and Run the Update

Follow these steps to run the update:

- a. Open “Vetcorder\_1\_9\_0\_Released.exe”
- b. Click the blue “Update” button

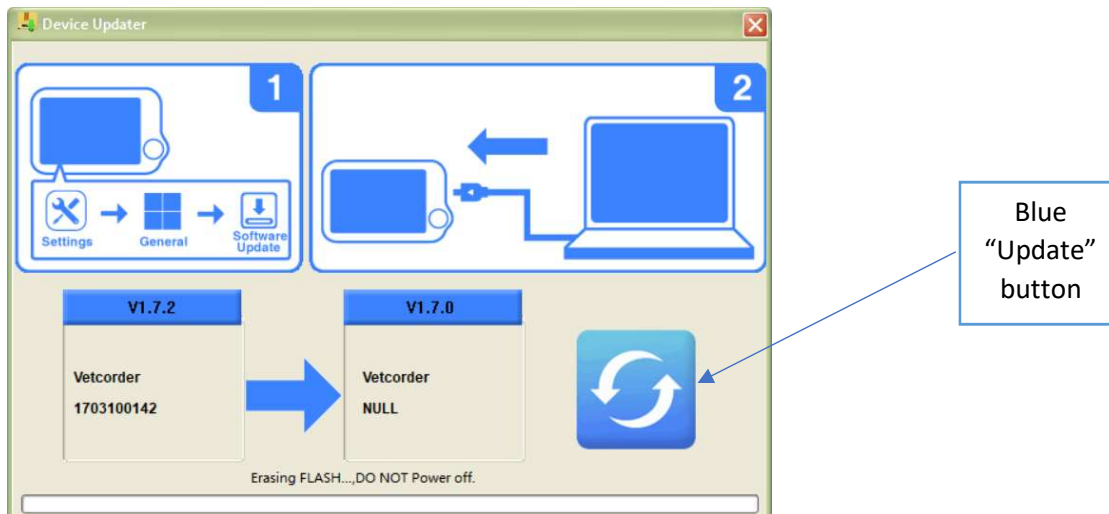

- c. If the update is working, you will see your Vetcorder’s serial number as shown here

## 4: Factory Reset Your Vetcorder & YOU ARE DONE!

After the update is complete, go to Settings, select General and then “Down” to select “Factory reset”. Once this completes there are a few basic setup steps and you are ready to go.

**IMPORTANT: Be sure to verify that the Bluetooth is set to ON if you use the Bluetooth capabilities of the device.**