



# Vetcorder Quick Start Guide

## Front View

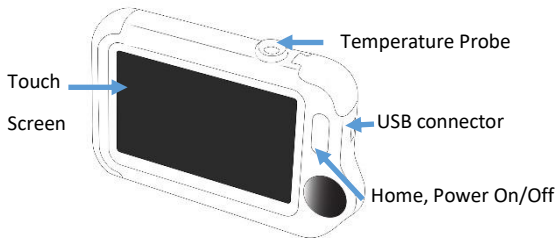

## Back View

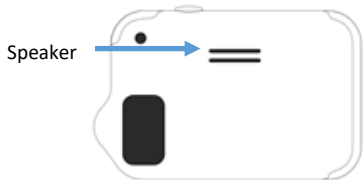

## Turning On & Off Vetcorder

- 1 Hold down the "Home, Power On/Off" button


### Note:

- Hold power button for 5 seconds to turn off


## ECG Recorder



- 1 Touch the "ECG Recorder" icon.
- 2 Choose ECG measurement mode: Lead I or Lead II

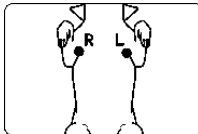

Lead I

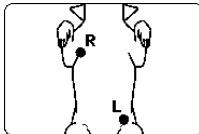

Lead II

- 3 Press the "play" button at the bottom-middle of the screen to take a 30-second recording (default)

## Pulse Oximeter




- 1 Touch the "Pulse Oximeter" icon.
- 2 Attach Probe to desired measurement site
- 3 Press the "play" button at the bottom-middle of the screen to take a 30-second recording

## Monitor Vitals



- 1 Touch the "Monitor Vitals" icon.
- 2 Connect cables via adapter.  
Note the ECG and SpO<sub>2</sub> ports
- 3 Simultaneous ECG and SpO<sub>2</sub> will display

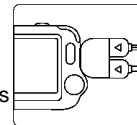


## Monitor Vitals (Connect to Tablet)



- 1 Install "Vetcorder Monitor" from App Store or Google Play on Tablet (must have Bluetooth 4.0 or above)

- 2 Connect vital monitor cable to Vetcorder

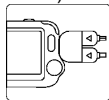

- 3 Touch the "Monitor Vitals" icon on the Vetcorder. At the same time, open the "Vetcorder Minimonitor" App on your tablet.

- 4 On the tablet, choose the Vetcorder to pair. The last 4 digits of the Vetcorder's serial number can be found on the back of the Vetcorder.

- 5 After connection, waveform and data will display on both Vetcorder and Tablet.

### Note:

- Make sure Tablet supports Bluetooth 4.0 or above.
- Use code **8888** the first time you pair a Vetcorder

## Thermometer



- 1 Touch the "Thermometer" icon.

- 2 Place sensor in ear



- 3 Click home button, and hold for around 3 seconds until you hear a long "Bi" beep.

- 4 Take down the device, and the screen shows the measurement result.




## Daily Check



- 1 Touch the "Daily Check" icon

- 2 Choose Pet or press "+" to create a new file

- 3 Connect ECG Clips and SpO2 Sensor and keep animal still.

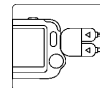

- 4 Once the signal is stable, a 20-second reading will be taken and stored. Summary results will display upon completion of the reading.

### Note:

- If Lead I signal is small, use Lead II for Daily Check measure.



Sentier HC, LLC  
161 W. Sunset Drive, Suite 102  
Waukesha, WI 53189  
(844) 838-2673

[www.SentierConnect.com](http://www.SentierConnect.com)  
[info@sentierconnect.com](mailto:info@sentierconnect.com)