



Vetcorder AirMate Quick Start Guide




1. Alarm Pause Button
2. Main Interface Button
3. Menu Button
4. Battery Indicator
5. Patient ID Number
6. Display Screen

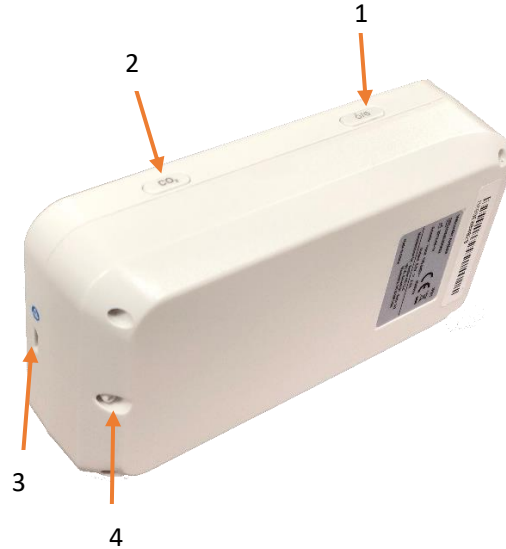

1. Power Button
2. Pause Button
3. USB Charging Port
4. Vent

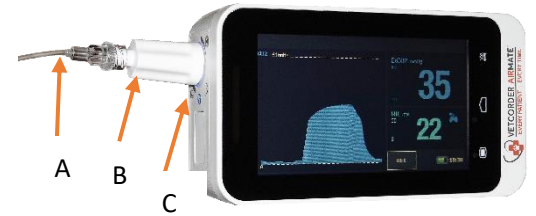

Connect Luer to Luer Sample Tubing (A) to the Moisture Filter (B) and connect the Moisture Filter to the AirMate (C) by pushing in and turning clockwise.

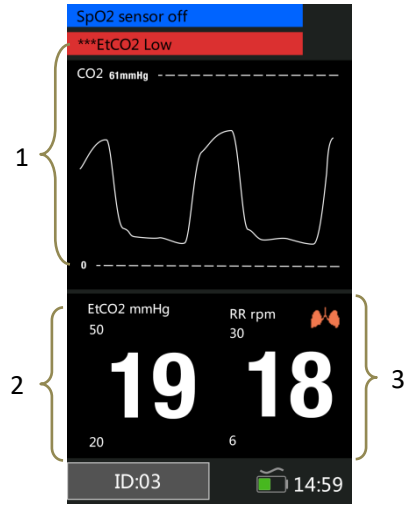

1. Current CO2 waveform display
2. CO2 parameter with HI/LO alarm setting
3. RR parameter with HI/LO alarm setting

CO2 Setup		
	CO2	RR
Alarm	Mid	High
Uplimit	42	30
Downlimit	32	8
Speed	6.25mm/s	
Unit	mmHg	
Scale	61mmHg	
Apnea Alm	Off	
OK	Cancel	

1. Press Menu button to get to CO2 Setup
2. Touch to right of Alarm to select "Mid/High"
3. Set your parameters. Uplimit should always be greater than Downlimit

Alarm level	Visual prompt
High	Alarm lamp flashes in red with 2 Hz.
Medium	Alarm lamp flashes in yellow with 0.5 Hz.
Low	Alarm lamp lights in yellow without flashing.



Sentier HC, LLC
161 W. Sunset Drive, Suite 102
Waukesha, WI 53189
(844) 838-2673

www.SentierConnect.com
info@sentierconnect.com