

## Let's download a recording from your Vetcorder!

By downloading and using the Vetcorder Connect App we'll be able to download either a "Daily Check" recording or an "ECG Recording"

On your Vetcorder's Second Screen (you can get there by pressing the flashing arrow on the right of the home screen) Press the "Upload" icon. (NOTE: on some older models this icon may be a "Connect" icon.) If you're using the Monitor Vitals App on the Vetcorder this must be closed before using the Vetcorder Connect App.



Select the Vetcorder App on your phone or tablet (This is the app with the link in the middle and named "Vetcorder Connect") **After a few seconds** you should see your phone or tablet trying to connect to your Vetcorder - you'll see this by noting the last four digits of the SN on the back of the Vetcorder will be displayed on the phone or tablet. Let's select that! (NOTE: if this is the first time connecting you may have to enter "8888" to continue.)

Once your Vetcorder connects to the app, you can download any recording available on that Vetcorder. The following will detail downloading a "Daily Check" or an "ECG" recording. **Please visit our Support area of [www.sentierconnect.com](http://www.sentierconnect.com) for detailed videos, OR contact us at 844-VETCORDER for a live demonstration of this feature.**

### DOWNLOADING A DAILY CHECK

While connected via Bluetooth and inside the Vetcorder Connect App, on the main screen (blue) you will see the last patient that used the Daily Check. To select a named patient click on the "pet" icon in the upper right hand corner. Let's go there! Now we're at a menu that displays the recorded Daily Checks available on the Vetcorder. An arrow pointing down means that would be a new download and an arrow pointing right means download has been completed. Find the Daily Check and select it's down arrow to start the download. once downloaded you will be able to select it by tapping it. This will bring us to the download screen where we can share or save the recording.

### DOWNLOADING AN ECG RECORDING

While connected via Bluetooth and inside the Vetcorder Connect App, on the main screen (blue) you will see three bars at the top left of the home screen. Let's go there! Once inside, we'll be able to select "ECG Recorder" Let's select that one! Now we're at a menu that displays the recorded ECGs available on the Vetcorder and arrows next to them for download. An arrow pointing down means that would be a new download and an arrow pointing right means download has been completed. Select your download based on timestamp. Selecting the down arrow will start the download of that ECG - once downloaded you will be able to select it by tapping it. This will bring us to the download screen