

Monitor Vitals (Connect to a Phone or Tablet)

Install "Vetcorder Monitor" from App Store or Google Play on a tablet or phone (must have Bluetooth 4.0 or above)

Connect vital monitoring sensors to the Vetcorder Pro

Touch the "Monitor Vitals" icon on the Vetcorder Pro. At the same time, open the "Vetcorder" App on your tablet or phone

On the tablet or phone, choose the Vetcorder Pro to pair. The last 4 digits of the Vetcorder Pro's serial number can be found on the back of the Vetcorder Pro

After connection, waveform and data will display on the Vetcorder Pro and the tablet or phone

Make sure the tablet or phone supports the Bluetooth 4.0 or above

Use code 8888 the first time you pair a Vetcorder Pro to a tablet or phone

Daily Check (Guest or create a patient)

Touch the "Daily Check" icon

Choose Guest or press "+" to create a new file

Connect ECG, SpO2 and Temperature Sensors and keep animal still

Once the signal is stable, a 30 second reading will be taken and stored. Summary results will display upon completing the reading.

If working with a smaller patient you may need to use the Lead II placement to obtain a better waveform

Large Animal Kit Available

If you're working with large animals in clinic or in the field, we have the necessary sensor and extensions that compliment the Vetcorder Pro. Please visit our "SHOP" page at www.sentierconnect.com or contact us by calling 844-VETCORDER.

Vetcorder Pro Quick Start

Front View



* unit cannot be charged while in use.

Back View



Power On & Off the Vetcorder Pro

Hold down the "Home, Power On/Off" button

Pulse Oximeter



Attach sensor cable to desired measurement site

Touch the "Pulse Oximeter" icon

Internal Temperature



Insert the internal temperature probe. The temp probe can be used rectally or via the esophagus for continuous monitoring

Touch the "Temperature" icon

Monitor Vitals



Connect cables to ports on side of unit

Touch the "Monitor Vitals" icon


Simultaneous ECG, SpO2 and Temp will display

ECG Recorder




Touch the "ECG Recorder" icon

Choose the ECG Mode: Lead I or Lead II

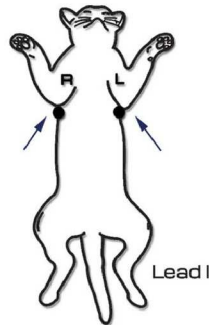

Lead I

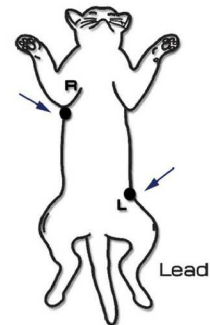

Lead II

Press the "play" button at the bottom-middle of the screen to take a 30-second recording. Settings can be changed to increase this time to 60 or 90 seconds.

Working with pocket pets, exotics or birds?

If you're working with pocket pets, exotics or avian we offer ZUMAYA needle electrodes for that difficult to obtain ECG reading. Visit our "ZUMAYA" page on our website at www.sentierconnect.com or contact us by calling 844-VETCORDER

Sentier Support is a click or call away

Every Vetcorder Pro comes with Sentier's World Class Support Staff. Just contact us by calling or clicking the support button on our website and we will return your call. REAL HUMANS, not recorders. EVERY VETCORDER PRO also comes with a complimentary in-service on the unit via phone or virtual visit.



VETCORDER™ PRO
EVERY PATIENT | EVERY TIME

Sentier HC, LLC
161 W. Sunset Drive, Suite 102
Waukesha, WI 53189

www.sentierconnect.com
info@sentierconnect.com