



VETCORDER AIRMATE™
EVERY PATIENT | EVERY TIME



Care and storage of your VetCorder Airmate™

As with all equipment, proper care, and maintenance with the VetCorder Airmate™ will prolong the life of the unit. In this segment, we will share a few tips on care and storage.

CLEANING YOUR AIRMATE

DO

- ✓ Wipe the device with a clean dry cloth
- ✓ Use a dab of alcohol on a soft cloth
- ✓ Gently install filter by slightly pushing down and turning clockwise
- ✓ Gently remove filter by twisting counterclockwise
- ✓ Keep the unit clean and dry

DON'T

- ✗ Submerge your Airmate in liquid
- ✗ Spray ANYTHING on your device
- ✗ Attempt to take device apart for service
- ✗ PUSH or force the filter onto the device



STORING YOUR AIRMATE

DO

- ✓ Remove tubing and filter
- ✓ Store cords LOSELY in a bag or on a tray
- ✓ Stock additional filters / tubing
- ✓ Stock low dead space adapters

DON'T

- ✗ Pull filter out of unit
- ✗ Throw or drop your Airmate
- ✗ Twist or pull charging cords



Please contact us at **844-VETCORDER** or email us at info@sentierconnect.com if you have **ANY QUESTIONS** on how to properly care and store your VetCorder Airmate!