



## Care and storage of your Vetcorder™ Classic

As with all equipment, proper care and maintenance with the Vetcorder™ Classic will prolong the life of the unit. In this segment, we will share a few tips on care and storage.



## CLEANING YOUR CLASSIC

### DO

- ✓ Wipe the device with a clean dry cloth
- ✓ Use a dab of alcohol on a soft cloth
- ✓ Gently remove ECG clips for cleaning
- ✓ Clean ECG buttons with a cotton swab

### DON'T

- ✗ Submerge your Vetcorder in liquid
- ✗ Spray ANYTHING on your device
- ✗ Drop or throw the Vetcorder Classic
- ✗ Put tension on the adapter or cords

## STORING YOUR CLASSIC

### DO

- ✓ Remove cables by base of connectors
- ✓ Remove clips from patients by pinching the clips to open and remove
- ✓ Store cords LOSELY in a bag or on a tray
- ✓ Leave the sensors plugged into the adapter
- ✓ Unplug the adapter by pulling straight out



### DON'T

- ✗ Wiggle or twist cords
- ✗ Pull by wires to remove from patients
- ✗ Wrap or hang cords for storage

Please contact us at **844-VETCORDER** or email us at [info@sentierconnect.com](mailto:info@sentierconnect.com) if you have **ANY QUESTIONS** on how to properly care and store your Vetcorder™ Classic!