



Care and storage of your VetCorder™ Pro

As with all equipment, proper care, and maintenance with the VetCorder™ Pro will prolong the life of the unit. In this segment, we will share a few tips on care and storage.

CLEANING YOUR PRO

DO

- ✓ Wipe the device with a clean green cloth
- ✓ Use a dab of alcohol on a soft cloth
- ✓ Gently remove ECG clips for cleaning
- ✓ Clean ECG buttons with a cotton swab

DON'T

- ✗ Submerge your VetCorder Pro in liquid
- ✗ Spray ANYTHING on your device
- ✗ Remove the case often
- ✗ Drop or throw the VetCorder



STORING YOUR PRO

DO

- ✓ Remove cables by base of connectors
- ✓ Remove clips from patients by pinching the clips to open and remove
- ✓ Store cords LOSELY in a bag or on a tray

DON'T

- ✗ Wiggle or twist cords
- ✗ Pull by wires to remove from patients
- ✗ Wrap cords for storage
- ✗ Hang cords on hooks for storage



Please contact us at **844-VETCORDER** or email us at info@sentierconnect.com if you have **ANY QUESTIONS** on how to properly care and store your VetCorder Pro!