



ZUMAYA™ ECG Needle Electrodes

Price List

Set of THREE - \$99 (NOTE: When ordering ZUMAYA™ please reference MAI Animal Health Item #48546)

- Three Pairs of ZUMAYA™ ECG Needle Electrodes

Recommendation

Sentier recommends having multiple sets available to ensure sterilized pairs are ready for use when needed.

Quick Guide

Proper Placement

- You will receive the best waveforms by inserting subcutaneously above both sides near the pectoral muscles on most small animals. Please contact Sentier for an Avian usage placement diagram.
NOTE: Subcutaneous insertion is sufficient.
- No liquids or gels are needed when using the ZUMAYA™ ECG Needle Electrodes.

Range of Patients

- ZUMAYA™ ECG Needle Electrodes may be used on any avian species from a finch up to a stork or ostrich.
- ZUMAYA™ ECG Needle Electrodes may be used on most exotic, rodent and reptilian species as well.

Recommended Number of Uses

- ZUMAYA™ ECG Needle Electrodes are “Disposable”- Single use only.

Sterilization Methods Reported

- Clinics have reported using gas sterilization or soaking in alcohol overnight.

ZUMAYA™ Uses in Clinic

- Compatible with CT Scan.
NOTE: NOT Compatible with MRI.
- Compatible with Radiography.
NOTE: Needles will appear in the x-ray; ensure placement does not interfere with the image area you are trying to capture.

Best Practices

- ZUMAYA™ ECG Needle Electrodes in conjunction with the VetCorder™ Pro are most effective during short procedures, collecting samples, radiographs and other transport situations.

Heart Rate Range

- The VetCorder™ Pro monitors 30 BPM to 350 BPM.

