

CONGRATULATIONS on your purchase of the Pulse Companion Digital Doppler by VetCorder!



LET'S UNPACK THE BOX

- (1) Pulse Companion Doppler
- (1) Pulse Companion flat probe
- (1) Sphygmomanometer
- (5) Blood pressure cuffs sized 1-5 to fit most small animals
- (1) Set of headphones
- (1) Bottle of ultrasound gel
- (1) Hard carrying case for transport and storage
- (1) USB Charging cord and block



PN Reference Below

Item	Product Description	UOM
48580	VetCorder Pulse Companion™ with Flat Probe Option	Each
48590	VetCorder Pulse Companion™ with Pencil Probe Option	Each

HOW TO USE THE PULSE COMPANION

- To power on the Pulse Companion, click the power / home button. To power the unit off, press and hold the power / home button for 5 seconds then press the confirmation on the touch screen
- Swab the pulse site with alcohol or warm soapy water (removing oil / dirt) it is often not necessary to shave the pulse site.
- Fit blood pressure cuff using the guide on the cuff for measurement (NOTE: too big or too small will hinder a good reading)
- Apply ultrasound gel to the pulse monitoring site (NOTE: using a good amount of gel on pulse site will assist in getting a stronger reading)
- Listen for the pulse then continue to hold probe over pulse site or tape in place while monitoring (NOTE: be careful not to apply too much pressure with holding in place as this could occlude the artery we're listening to)
- With the sphygmomanometer connected to the cuff, place the cuff around the desired limb and inflate the cuff until the pulse sound is no longer heard
- Slowly release the pressure in the cuff by using the toggle device found on the back of the sphygmomanometer
- When the first sound is heard from the Pulse Companion Doppler note the reading from the dial of the sphygmomanometer - repeat this procedure 3 x to gain an accurate interpretation - this is your patient's systolic BP reading (NOTE: while it is possible to detect a diastolic pressure it is not often heard on some patients)



Don't forget to register your warranty
844-VETCORDER

www.sentierconnect.com/product-warranty-registration

844-838-2673

www.sentierconnect.com/pulse-companion
Copyright 2024. All rights reserved.